

Abstract

Pediatric complete tibia fractures are common and are currently managed non-operatively by casts; however, a surgically implanted device would provide more structural stability, expedite bone healing, and limit uncomfortable casting for the patient. Elastic nails are now used to surgically fix such fractures, but do they not provide rotational fixation or sufficient stabilization of non-medial fractures. Insufficient stabilization and/or axial rotation within the bone could lead to device failure, unnecessary pain, and corrective surgery. This semester's goal is to design a device than will incorporate a metal biaxial braid to provide bending and rotational stiffness for the fractured bone. The device must be flexible for 45° insertion into the intramedullary canal of the tibia without disturbing the epiphyseal growth plates. A nut above a free-sliding top cap is twisted down a threaded K-wire centerpiece with the use of a flexible drive shaft, which moves the top cap toward the bottom cap and compresses the surrounding metal biaxial braid. This braid then expands, pushing against the canal wall and stabilizing the fracture. Our data show that this new device has a higher rotational stiffness, but a lower bending stiffness compared to elastic nails. Future work includes improving the mechanical properties of the braid by altering the braid structure and eliminating the use of non-biocompatible components during fabrication to advance this device toward a clinical setting.

Background

Motivation & Current Devices

Problem Motivation

•~3,100 pediatric bone fractures occur each year²

- •15% of pediatric bone fractures are tibia fractures²
- •5% of pediatric tibia fractures cannot be addressed by casting
- •Weight bearing bone requires proper healing for development^{3,4} •Epiphyseal growth plates at proximal and distal ends of tibia⁵

Current Devices

Elastic Nails: •Two bent titanium nails inserted into the side of the bone⁵

Figure 2. Elastic nails implanted in pediatric femur. The location of implantation is similar for the tibia. Implantation avoids growth plates.⁵ •Most common technique for pediatric tibia fractures

• Avoids growth plates⁶

•Problems: Surface contact restricted to mid-point of bone • No rotational fixation⁷

Previous Semester Progress

Braided Cylinder: •Stainless steel biaxial braid •Axial compression \rightarrow Radial expansion • surface area results in \uparrow axial fixation



Figure 3. Images of stainless steel biaxial braid.^{8,9}

Problem Statement

- Device must provide structure and stability for complete pediatric tibia fractures
- Avoid epiphyseal growth plates
- This semester:
 - Design device to implement metal biaxial braid in bone • Increase axial fixation compared to elastic nails

Design Specifications

- Limit axial rotation
- Rigid to stabilize fracture
- Implanted in the intramedullary canal avoiding growth plates (45° angle, 8 mm)
- Implanted for 6-9 months
- Visible on x-ray for implantation procedure
- Biocompatible and complies with FDA implant guidelines

A NOVEL FIXATION DEVICE FOR PEDIATRIC ORTHOPAEDIC TIBIA FRACTURES

<u>TEAM</u>: Evan Lange, Karl Kabarowski, Tyler Max, Sarah Dicker <u>CLIENT</u>: Matthew Halanski, MD, School of Medicine and Public Health, Department of Orthopedics and Rehabilitation – University of Wisconsin - Madison ADVISOR: Professor Paul Thompson, PhD, Department of Biomedical Engineering – University of Wisconsin - Madison



Figure 1. X-ray of pediatric complete tibia fracture.¹

• Flexible device to implement braided cylinder

- Centerpiece
- Threaded Kirschner wire (K-wire) Braid Connection
 - Free-sliding top cap
 - Fixed bottom cap (fixed with nail)
- Metal biaxial braid
- Method of Compression
 - Threaded nut above top cap
- **Device to drive nut for compression**
 - External hex broach
 - Manual auger
 - Flexible hollow drive shaft

Bending stiffness:

 $k = M/\theta$

- M = moment θ = deflection angle
- Approach:
- Cantilever loading • Determine moment at fracture for various loads
- Measure deflection angle of fracture point when loaded using ImageJ
- Compute bending stiffness (moment/ deflection)
- **Conclusion:** Elastic nails have a greater bending stiffness than the braid device





Figure 5. Cantilever Bend Testing and Results –

significant (p < 0.01), confirmed by a t-test. Error bars: +/- 1 SEM



- Apply moment until 90° deflection
- Measure force with gauge
- Compute rotational stiffness (moment/deflection)
- **Conclusion:** The braid device has a greater rotational stiffness than elastic nails

Figure 6. Rotational Testing and Results –

A) Image of rotational testing setup. Sawbone model of fractured bone with braid device inserted is secured with a C-clamp. Load is applied via a moment arm at the free end of the bone. Measure the amount of force required to deflect 90°; B) Bar graph of mean rotational stiffness (in Ib/degree) for elastic nails and the braid device (n = 5). The force required to rotate elastic nails was not measurable using the available force gauge. The braid device has a mean rotational stiffness that is at least an order of magnitude greater than the elastic nails. Error bars: +/- 1 SEM

Elastic Nails

Braided Device





	Provider	Price
ot Pieces	Home Depot	\$9.70
aid	Wirecare.com	\$47.15
n Caps	Caspersen	\$669.00
oach	Polygon Solutions	\$391.61
	Ace Hardware	\$6.00
	Suhner Manufacturing Inc.	\$0.00
	Client: Dr. Halanski	\$0.00
		\$1,123.46
pet		\$3 876 54

[7] O'Brien, T., Weisman, D.S., Ronchetti, P., Piller, C.P., Maloney, M. (2004). Flexible titanium nailing for the treatment of the unstable pediatric tibial fracture. J Pediatr Orthop 24:601-609. [8] adapted from http://www.techflex.com [9] http://photography-on-the.net/forum/showthread.php?t=828639 ©2010 Jordan Steele

[10] http://www.deltasurgicalinstruments.com/catalog/k-wire.pdf

[11] http://suhner-transmission-expert.com/

[12] http://chinesefingertrap.co.uk/