

Problem Statement

A novel airway trainer device is needed for clinicians to practice intubation procedures. The custom trainer must reflect the complexity of a specific patient's airway anatomy for realistic simulation. These devices are critical to ensure adequate training for clinicians during difficult intubations.

Motivation/Background

Background

- Every year, close to 400,000 airway management procedures are performed to restore patient airway patency [1]
- Overall intubation success rate approximately 78.8%

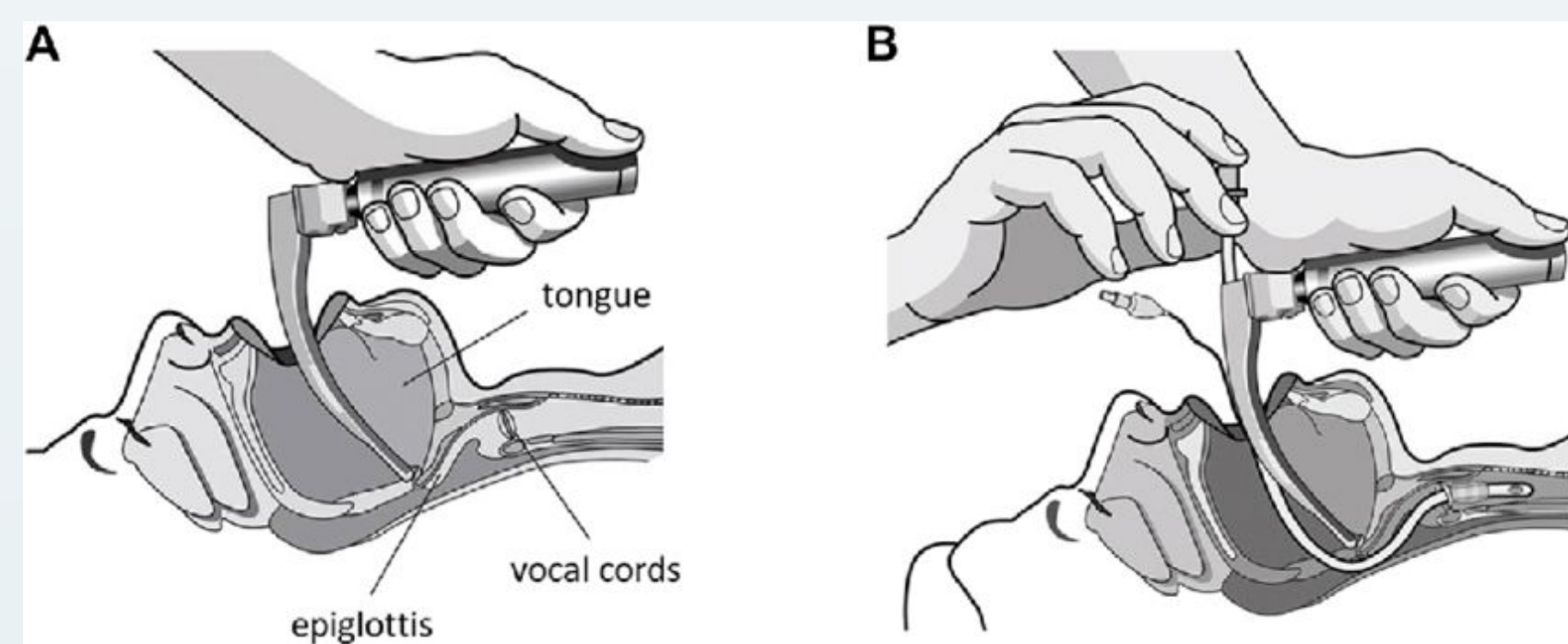


Figure 1: Intubation Procedure [2]

- Critical condition first-pass success is 68.5% [3]
- Complications due to patient abnormalities are rare (0.0015% of cases) but account for 25% of anesthesia related deaths [4]
- Extensive training via airway trainers required for first-pass success in "can't intubate, can't ventilate" situations where surgical interventions are the only next option

Competing Designs

- Popular Laerdal trainer offers healthy airway anatomy [5]
- Not reflective of patient specific pathologies (complications due to abnormal anatomy, edema, previous surgeries)
- Models are expensive (\$3,000 average) [6]

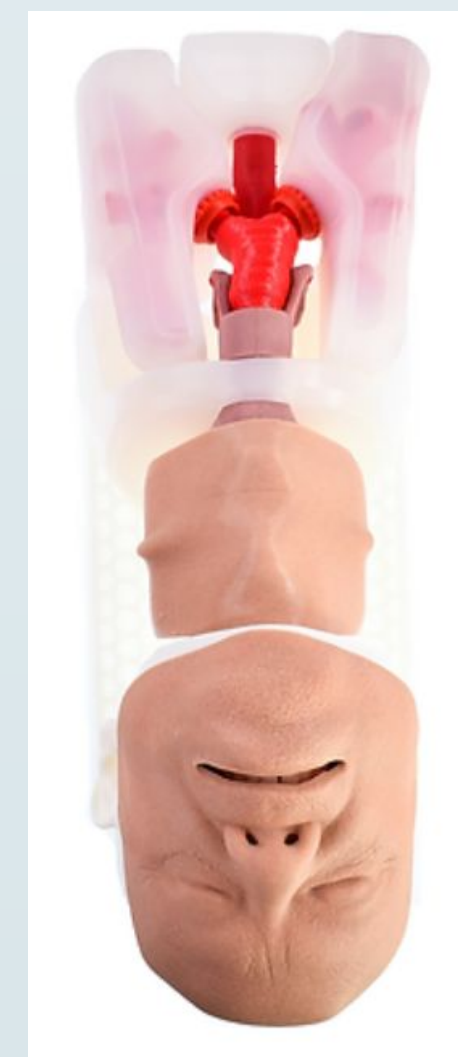


Figure 2: Decent Simulators Airway Trainer [7]

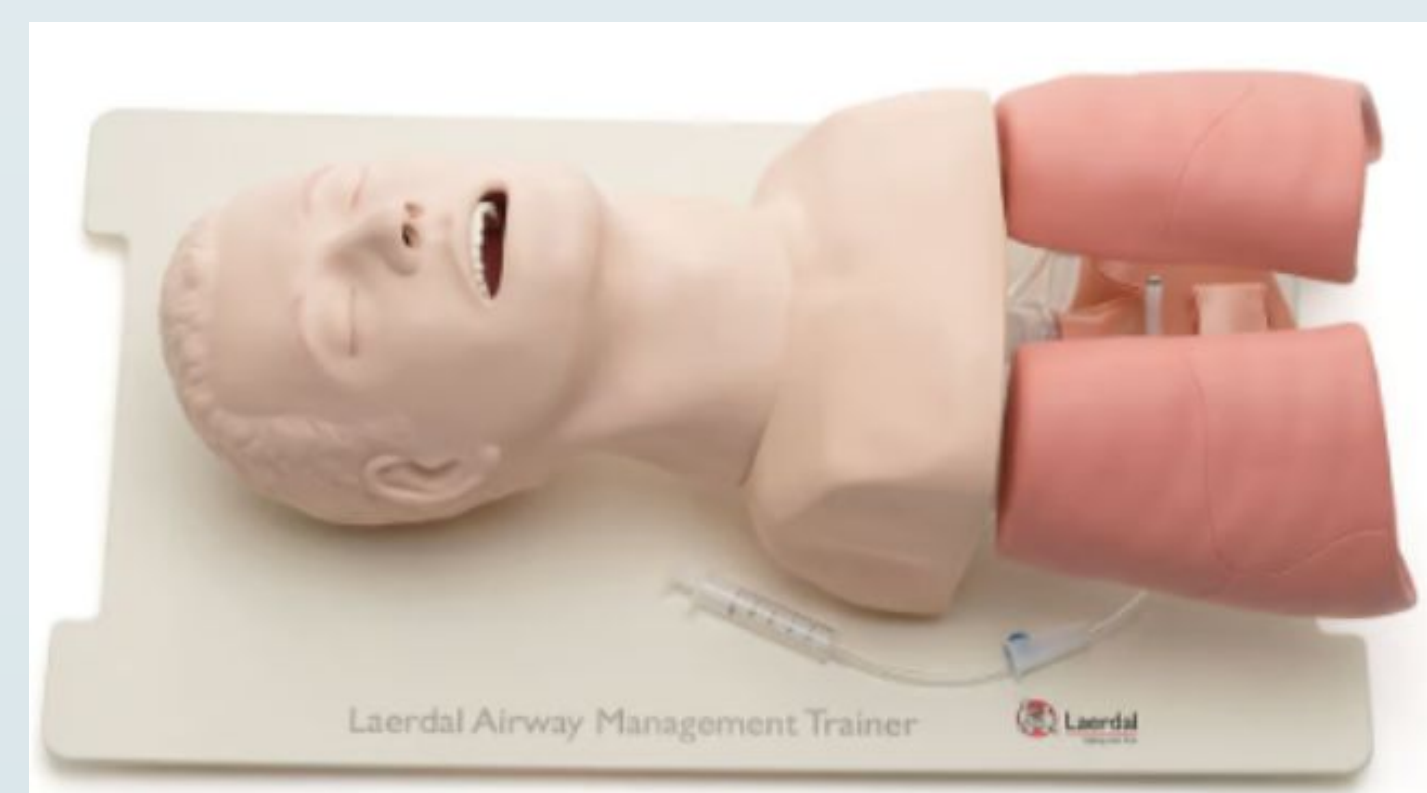


Figure 3: Laerdal Airway Trainer [5]

Design Criteria

- Swappable patient specific anatomy
- Durability measured by minimum Young's Modulus of 16 MPa \pm 8 MPa and Shore A hardness of 80 [8, 9]
- Material is water-based lubricant resistant
- Successful intubation at least 96.8% of the time [10]
- Budget of \$750
- Adaptable in less than one week lead time for new patient

Final Design

Final Design of Airway:

- Utilized auto segmentation capabilities in 3D Slicer
 - CT scans chosen due to availability and simplification
 - Eliminated manual-segmentation for model throughput
- Removed anatomy after primary bronchi
- Solid model representing inner wall of airway anatomy
- Post processing in OnShape to create "negative"



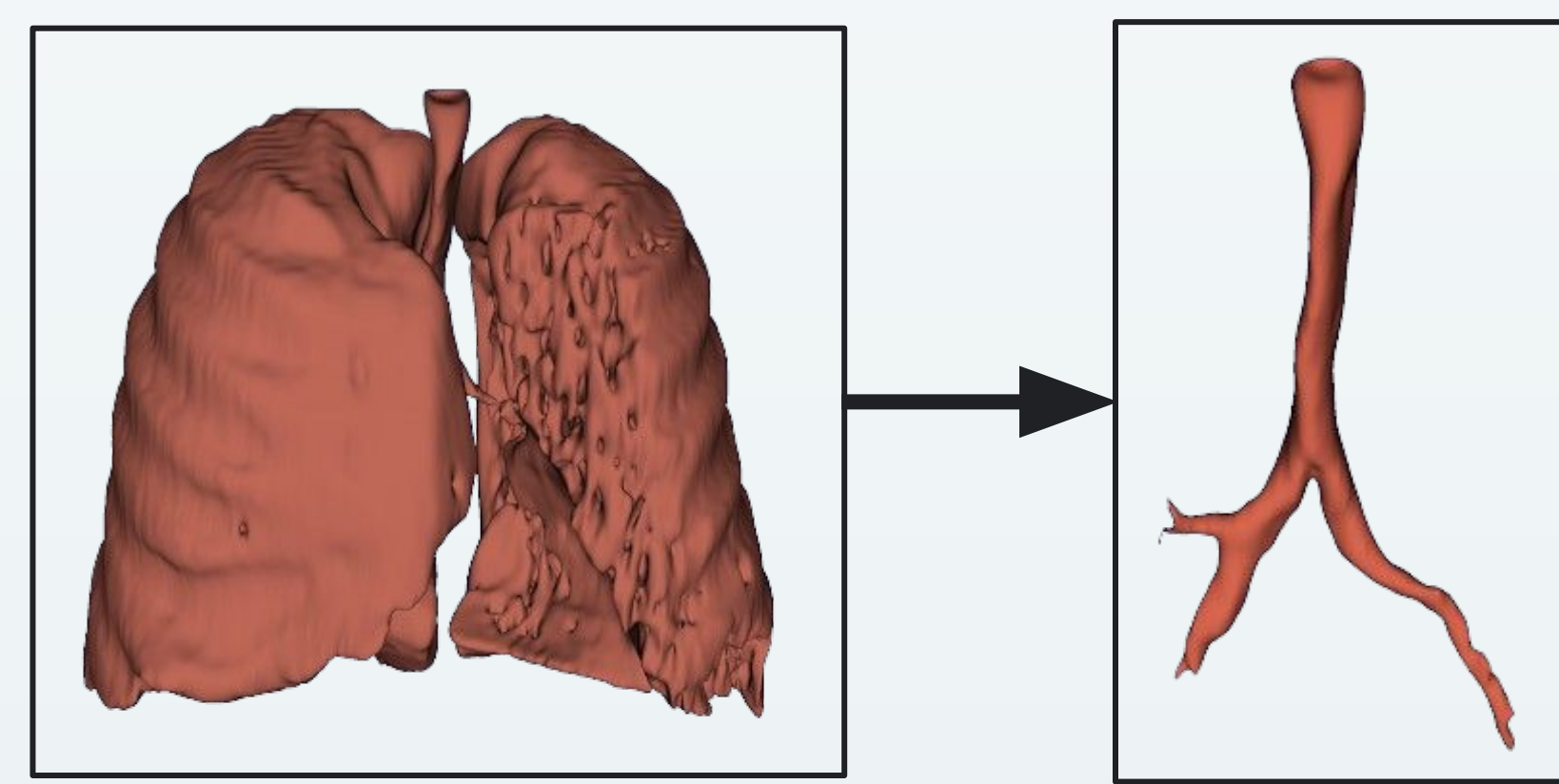


Figure 4: Model processing workflow

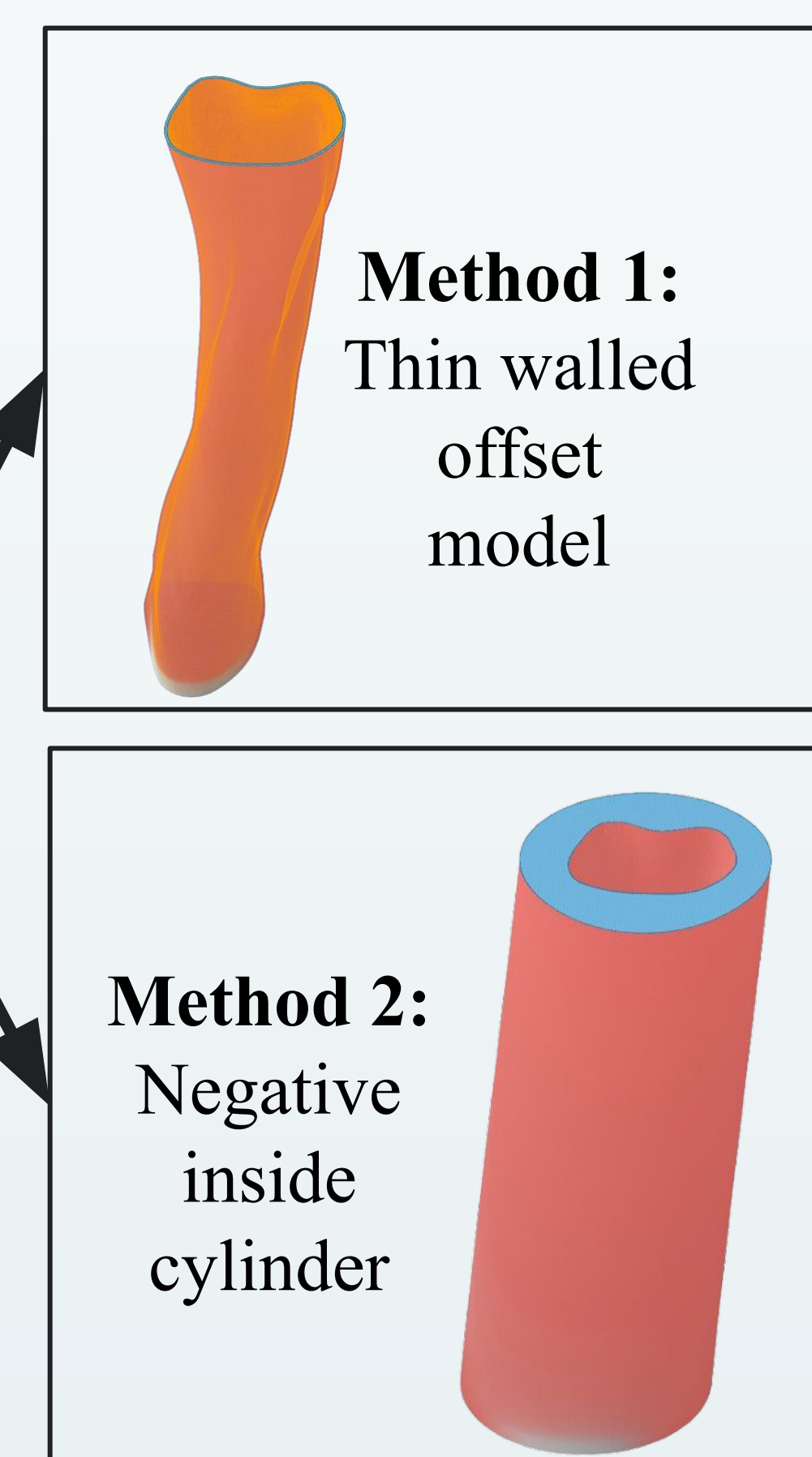

Two Paths for Model to Trainer:

- Difficult due to airway model representing air instead of tissue
- 1: Replicate thin tissue lining trachea using offset surfaces
 - Variable thickness per model
 - Constraints due to CAD
- 2: Subtract model from cylinder
 - Replicates anatomy internally
 - Thicker model for more accurate biomechanics of airway
- Both require experience in CAD
- No commercial software available to speed or simplify model creation
- Requires additional post processing to work with upper airway features

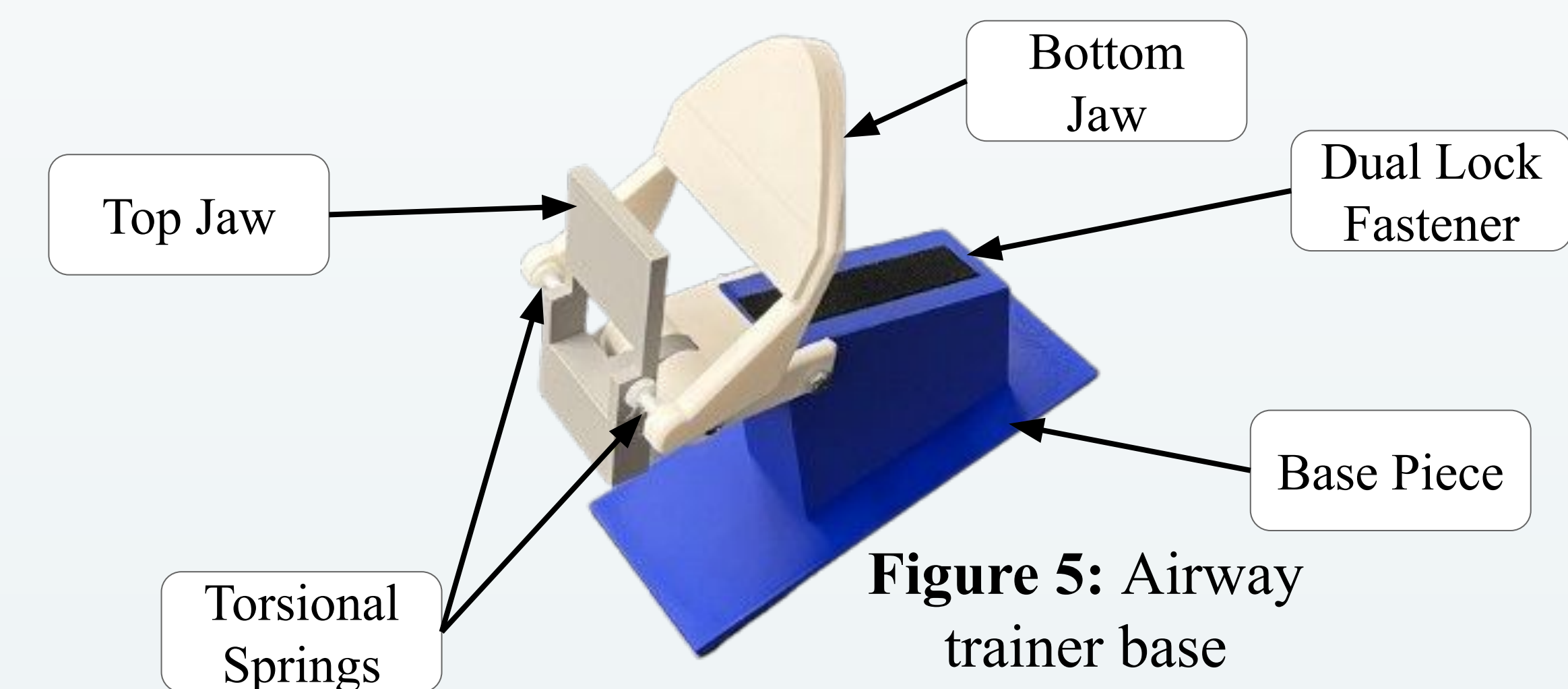

Final Design of Airway Trainer Base:

- Sloped platform to mimic anatomical airway placement
- TPU torsional springs that replicate physiologically accurate range of motion in the jaw
- Dual Lock fasteners used to allow for versatility in airways compatible with the design

Results and Discussion

Finite Element Analysis (FEA) Test

Goal: Determine if the pieces of the airway trainer base will fail under the force exhibited during intubation

- Maximum force during intubation is 63 N [11]
- Force applied to base pieces using SOLIDWORKS FEA software
- Factor of Safety (F.O.S.) \leq 5 was considered a failure
- Since Top Jaw and Bottom Jaw failed, modeling with stronger materials, such as ABS, should be considered

Area Force Applied	Base Piece	Neck Hinge	Top Jaw	Bottom Jaw
Top	✓	✓	X	X
Right	✓	✓	✓	✓
Both	✓	✓	X	X

Table 1: Results of FEA Test - Green indicates passing and red indicates failing


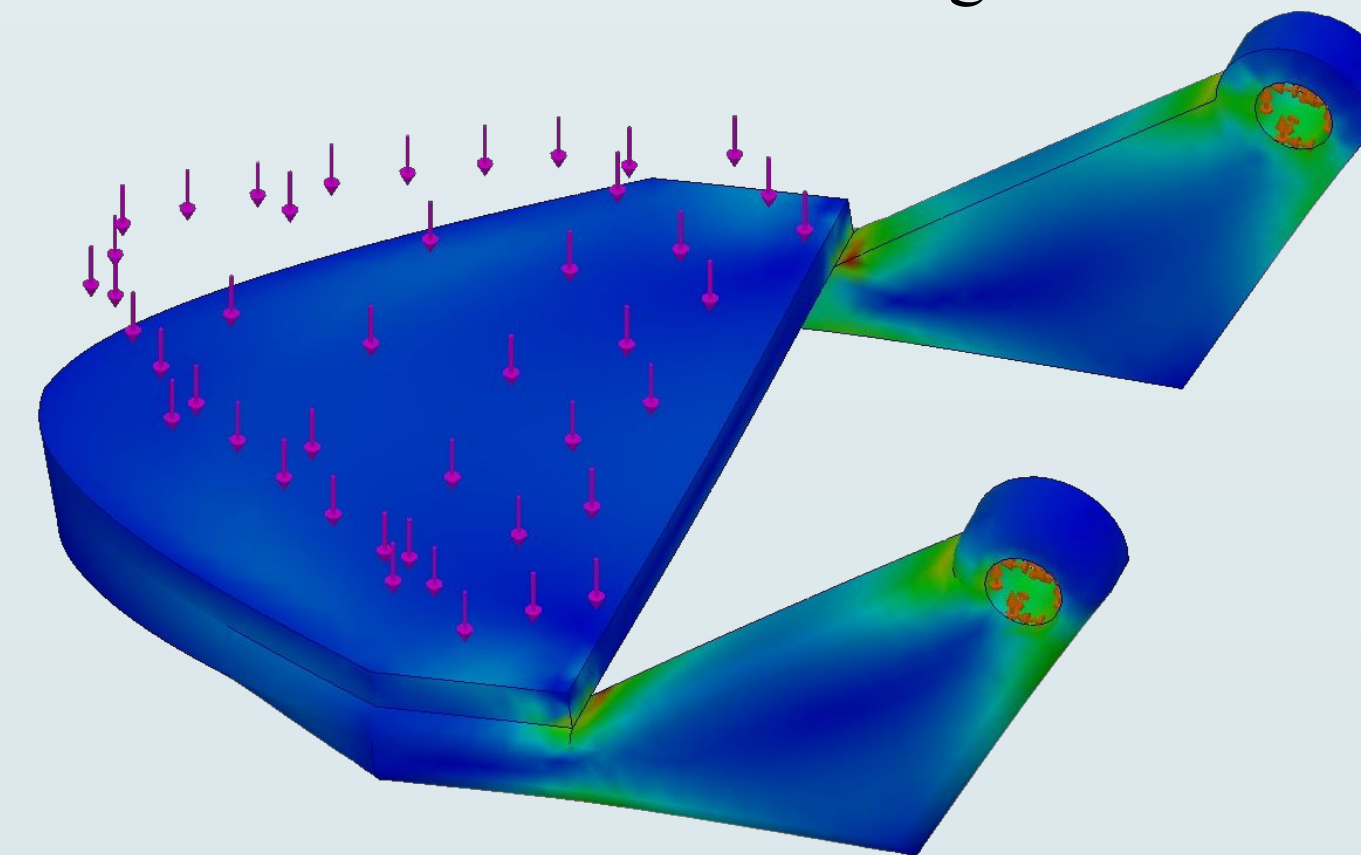


Figure 6: FEA Test on Bottom Jaw - Blue indicates low stress, green is moderate, and red is high

Figure 7: Graphed tensile testing data

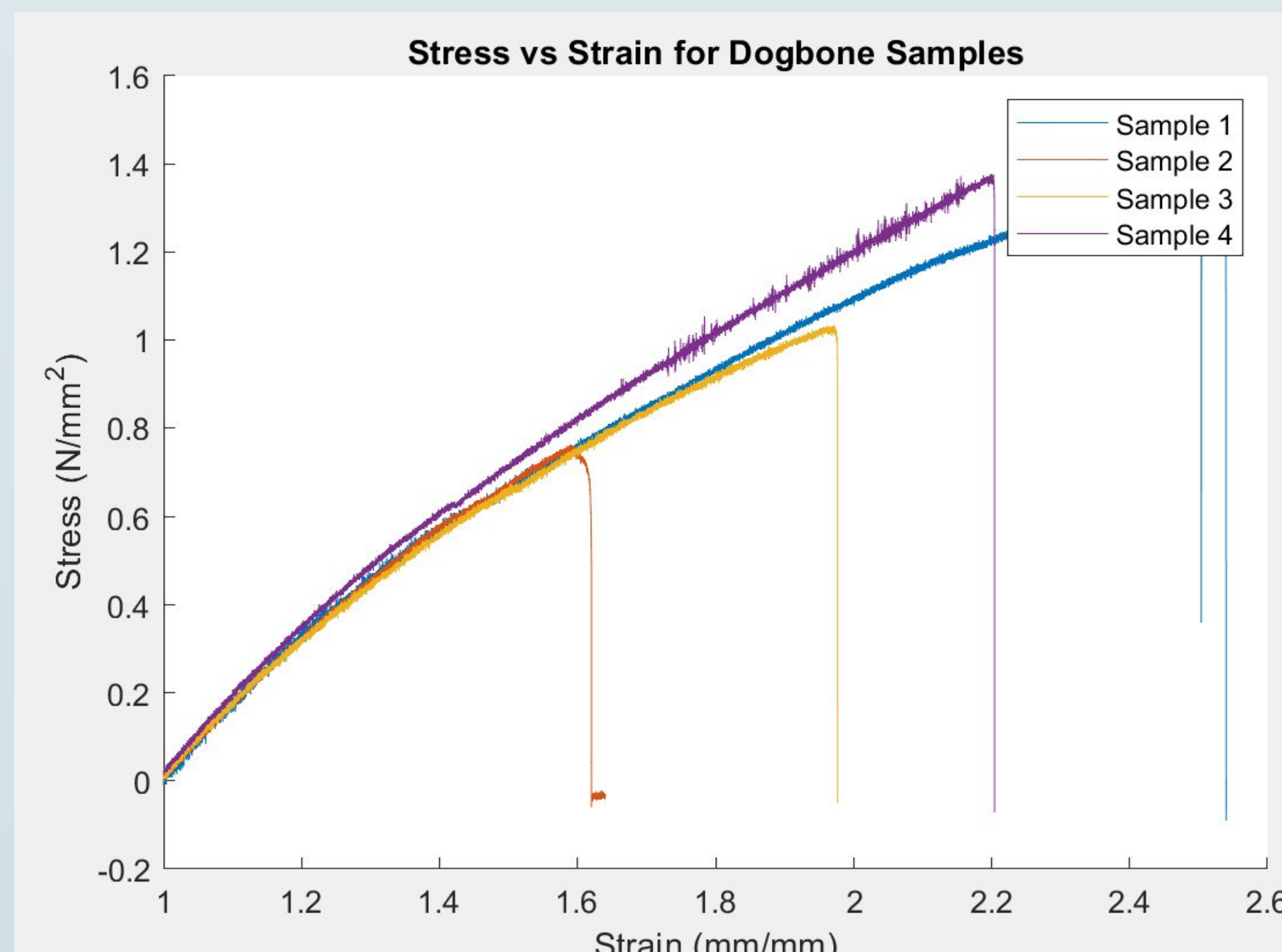

Neck Range of Motion (ROM) Test

Goal: Determine if the trainer accurately represents the ROM of a patient's neck

- Testing the modeling of the neck by two hinges
- Angles and measurements were taken from AirSim trainer prior to disassembly to compare values
- Limiting or increasing ROM will decrease realism
- Incorrect ROM can lead to harm caused to patient because of clinician training on the model
- AirSim: 50°, 45°; Our trainer: 55°, 27° (flexion and extension respectively)



Figure 8: Final hinge maximum flexion



Figure 9: Final hinge maximum extension

Material Tensile Test

Goal: Determine the Young's Modulus (YM) of the Formlabs 50A resin to ensure proper biomechanical properties and realism in trainer

- Significant support required leads to variability in material strength
- YM calculated from tensile testing of ASTM-D838 Type IV Dogbone sample and compared to datasheet from Formlabs
- Calculated Young's Modulus: 0.91 MPa, Stated as 1.59 MPa [12]
- Differences arise from support removal harming model

Future Work

- Collaborate with client to ensure prompt access to scans
 - Receive IRB approval or dataset access
- Work with Radiology for segmentation support
- Explore Materialise availability inside UW Health [13]
- Utilize MeshMixer to process STL files [14]
- Improve the trainer's facial anatomy to allow for compatibility with NPAs and other advanced airways [15]
 - Create mouth, nose, and teeth for realistic intubation
- Contact 3D printing companies to explore improvements in model production to eliminate client processing prints

Acknowledgements

BME Department
 Dr. Kristopher Schroeder
 Dr. John Puccinelli
 Srihari Gopalan

Makerspace Staff
 Jesse Darley
 Chris Meyer, Sector67

References

[1] S. Maguire, P. R. Schmitt, E. Sternlicht, and C. M. Kofron, "Endotracheal Intubation of Difficult Airways in Emergency Settings: A Guide for Innovators," *Med Devices (Lond)*, vol. 16, pp. 183–199, Jul. 2023, doi: 10.2147/MDER.S419715.

[2] I. Matchaia-Morin et al., "Design and Additive Construction of a Video-Laryngoscope for Endotracheal Intubation of Adult Patients," *Front. Mater.*, vol. 9, May 2022, doi: 10.3389/fmats.2022.906851.

[3] J. Thomas et al., "Association Between Emergency Medical Service Airway Intubation Rate and Intubation Success," *Ann Emerg Med*, vol. 84, no. 1, pp. 1–8, Jul. 2024, doi: 10.1016/j.annemergmed.2023.11.005.

[4] M. J. Cook and S. R. MacDougall-Davis, "Complications and failure of airway management," *British Journal of Anaesthesia*, vol. 109, pp. i68–i85, 2012, doi: 10.1093/bja/aes393.

[5] "Laerdal Airway Management Trainer," Laerdal Medical, Accessed: Dec. 05, 2024. [Online]. Available: <https://laerdal.com/distributors/products/skills-proficiency/airway-management-trainers/laerdal-airway-management-trainer/>

[6] "AirwayMan Airway Management Trainer," Simulab Corporation, Accessed: Dec. 05, 2024. [Online]. Available: <https://simulab.com/products/airwayman-airway-management-trainer>

[7] "Airway management task trainer," Decent Simulators, Accessed: Dec. 05, 2024. [Online]. Available: <https://www.decent-simulators.com/product-page/airway-management-trainer>

[8] T. Agbiki et al., "Fabrication and assessment of a bio-inspired synthetic tracheal tissue model for tracheal tube cuff leakage testing," *BJA Open*, vol. 10, p. 100290, Jun. 2024, doi: 10.1016/j.bjao.2024.100290.

[9] I. Safishekan, M. Tafazzoli-Shadpour, M. Abdous, and M. B. Shadmehr, "Mechanical Characterization and Constitutive Modeling of Human Trachea: Age and Gender Dependency," *Materials (Basel)*, vol. 9, no. 6, p. 456, Jun. 2016, doi: 10.3390/ma9060456.

[10] Michèle Min Chan, Christophe Fehlmann, M. Pasquier, L. Suppan, and Georges Louis Savoldelli, "Endotracheal Intubation Success Rate in an Urban, Supervised, Resident-Staffed Emergency Mobile System: An 11-Year Retrospective Cohort Study," *Journal of Clinical Medicine*, vol. 9, no. 1, pp. 238–238, Jan. 2020, doi: 10.3390/jcm9010238.

[11] M. J. Bishop, R. M. Harrington, and A. F. Tencer, "Force applied during tracheal intubation," *Anesthesia & Analgesia*, vol. 74, no. 3, Mar. 1992, doi: 10.1213/0000539-199203000-00016.

[12] Formlabs, "Formlabs Data Sheet Flexible Filament 50A V1," Formlabs.com, Accessed: Dec. 05, 2024. [Online]. Available: <https://formlabs-media.formlabs.com/datasheets/2001420-TDS-ENUS-0.pdf>

[13] "Materialise Mimics | 3D Medical Image Processing Software," Accessed: Dec. 05, 2024. [Online]. Available: <https://www.materialise.com/en/healthcare/mimics-innovation-suite/mimics>

[14] "MeshMixer | Autodesk App Store," Accessed: Dec. 05, 2024. [Online]. Available: <https://apps.autodesk.com/FUSION/en/Detail/Index?id=4108920185261935100&appLang=en&os=Win64>

[15] DuCanto, J., Serrano, K., & Thompson, R. (2017). Novel Airway Training Tool that Simulates Vomiting: Suction-Assisted Laryngoscopy-Assisted Decontamination (SALAD) System. *Western Journal of Emergency Medicine*, 18(1), 117–120. <https://doi.org/10.5811/westjem.2016.9.30891>